
Mini Dome Switch #72S
(With Lights, Music & Vibration)

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Fun way to activate your devices!

Touch anywhere and see lights, spinning glitter and hear exciting music while activating your device. The Mini-Dome increases visual attention and is great for learning cause and effect. A special control allows the user to select any or all of the rewards. Can be used on a tabletop or mounted on a door or wall. Wall mounting bracket (sold separately) allows for attachment to walls and doors. See item #466 below. Size: 7¾"W x 4½"H. Requires 4 AA Batteries. Weight: 1 lb.

Operation:

1. The Mini Dome Switch requires four AA batteries (Not included). The battery compartments are located under the base of the unit. Carefully turn the switch over, and then remove the battery covers with a small Philips screwdriver. Install new batteries, being careful to observe proper battery polarity. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Replace the battery case covers and secure with the small screws. Do not over-tighten.
3. To turn on or off the desired stimuli (1-vibration; 2-music; 3-lights), push the small white rocker switches to the on or (open position). Open position means off. Please see Fig 1.

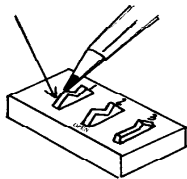


Figure 1.

4. Press anywhere on the switch and you will be rewarded with the selected stimuli vibration, lights & music.

5. To operate as a switch connect an external toy or device via the 1/8-inch cord jack on the unit. If you need to use a ¼" adaptor, it must be a mono adaptor, not stereo. Press anywhere on the switch to activate your toy or device. Once you release pressure from the switch your toy or device will turn off.

Important Notes:

When the Mini Dome is not in use, make sure that it is not being depressed in any way. Doing so will drain the batteries, even if you have the Lights, Music & vibration set to off. To prevent battery drain during storage, remove batteries.

Troubleshooting:

Problem: The Mini Dome Switch does not operate properly.

Action # 1: Make sure the batteries are in the battery compartment properly, and making good contact.

Action # 2: Make sure you have the desired stimuli selected as explained in operation No.3.

Action # 3: If the unit still does not work properly, replace the batteries.

Problem: The Mini Dome Switch does not activate a connected toy/device.

Action #1: Make sure all connections are tight.

Action #2: Check batteries in toy/device, replace if weak or dead.

Care of Unit:

The Mini Dome Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.