



enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Jumbo Switch (Green) #768

With Music, Lights & Vibration

USER'S GUIDE

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Stimulating feedback!

Our very sensitive, easy to use switch is now available with lights, music and vibration. It provides the user with great visual, auditory, and tactile stimulation. Has individual on/off switches for controlling the three stimuli. Size: 5"D x ½"H. Requires 2 AAA Batteries. Weight: ¾ lb.

Operation:

1. The Jumbo Switch requires 2 AAA batteries to operate (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Gently turn the unit over to face the black battery compartment cover. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and lift the cover off, it may be necessary to use the end of the screwdriver to lift one edge of the cover. Observing proper (+) & (-) battery polarity, install 2 AAA size batteries into the holder. Replace the battery compartment cover and screw.
3. To turn ON or OFF the desired stimuli (1-vibration; 2-music; 3- lights), push the small white rocker switches to the on or (open position). Open position means OFF. Please see Fig 1 and back of user's guide.

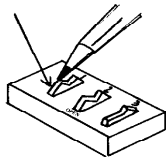


Figure 1.

4. Plug the Jumbo Switch into the jack on your switch-adapted toy/device.
5. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.

6. Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be no gaps.
7. Simply press anywhere on the Jumbo Switch's colored top to activate your toy/device.
8. The toy/device will only remain activated while the Jumbo Switch continues to be actuated. Once you release the switch, the toy/device will turn off. The user will also be rewarded with Lights, Music & Vibration (depending on the switch settings) as long as the switch is being activated. The Jumbo Switch can be used as a sensory stimulating toy on its own.
9. **Please Note:** For prolonged storage we recommend removing the batteries. This switch will not work if the batteries are removed, weak or dead. Batteries are needed for switch to work.

Troubleshooting:

Problem: The Jumbo Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Action #4: Make sure the batteries are fresh; this switch will not work without batteries.

Care of Unit:

The Jumbo Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.