
Button Click Switch #775

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

This switch hits the spot!

This reliable button switch provides momentary output using just 40 grams of pressure. Size: 2½"L x 2½"W. Weight: ¼ lb.

Operation:

1. The Button Click Switch does **not** require any batteries to operate.
2. Plug the Button Click Switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Button Click Switch and your toy/device is plugged in all the way. There should be no gaps.
6. Simply press anywhere on the Button Click Switch's colored top to activate your toy/device.
7. The toy/device will only remain activated while the Button Click Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Button Click Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Button Click Switch and your toy/device is plugged in all the way. There should be **no** gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Button Click Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a nontoxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.