



Photos 4 & 5

Care of Unit:

The Mounting Plate can be wiped clean with any mild household multi-purpose cleaner and disinfectant. **Do not use abrasive cleaners**, as they will scratch the surface of the unit.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com

Mounting Plate for Jumbo Switch #7808
Mounting Plate for Gumball Switch #7809

USER'S GUIDE



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Accessible!

Put your switch wherever you need it. Now you can mount your Gumball or Jumbo switch on any mounting arm or gooseneck. Simply attach your switch to the plate with the three screws included with the plate. Then attach the switch and plate to your mount using the included 1/4"-20 connection post. Size for Gumball Plate: 3"L x 3"W x 1/4"H. Weight: 1/2 oz. Size for Jumbo Plate: 6"L x 6"W x 1/2"H. Weight: 2 oz.

1. **Mounting Plate Switch Attachment:** Your mounting system consists of three main parts, a mounting plate, 3 screws to mount your switch to the plate and 1 1/4"-20 threaded post. The first thing you will need to do is with a Philips Head screw driver remove the 3 mounting plate screws from the plate (see photo 1). Next place your switch on the mounting plate and line up the three screw holes. Finally screw in all three screws to secure the switch to the mounting plate (see photos 2 & 3).



Photo 1



Photo 2



Photo 3

2. **Attaching Mounting Plate to Mounting Arm:** Now with your switch secured to the mounting plate you are ready to attach your switch to any mounting device that used the standard 1/4"-20 threaded system. If your mounting device does not have a male threaded post to connect our mounting plate you will need to use the included 1/4"-20 post (see photos 4 & 5 on back page).