



Sticker Placement Guide



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com

Jumbo Switch with Vibration #791

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

New feature!

Our popular Jumbo switch now has vibration. Press anywhere on the disk to feel vibration and to activate a device. Base can be removed if not needed. Includes two free clear covers for placing your icons: one with a lip and one without. Size: 5"D x ½"H. Requires 2 AAA Batteries. Weight: 1 lb.

Operation:

1. The Jumbo Switch requires 2 AAA batteries to operate (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Gently turn the unit over to face the black battery compartment cover. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and lift the cover off, it may be necessary to use the end of the screwdriver to lift one edge of the cover. Observing proper (+) & (-) battery polarity, install 2 AAA size batteries into the holder. Replace the battery compartment cover and screw.
3. Plug the Jumbo Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Jumbo Switch's colored top to activate your toy/device.
6. The toy/device will only remain activated while the Jumbo Switch continues to be actuated. Once you release the switch, the toy/device will turn off. The user will feel a gentle vibration from the switch as long as the switch is being activated.

7. **Please Note:** There is no ON/OFF switch for the vibration, keep this in mind when storing the switch. Never lay the switch face down or in a way that the switch plate will be depressed. Doing so will cause the batteries to go dead. For prolonged storage we recommend removing the batteries to prevent this from happening. This switch will not work if the batteries are removed, weak or dead. Batteries are needed for switch to work.

Mounting Plate Options:

The Jumbo Switch has a removable mounting base option. To use without the base, simply remove the two small Phillips Head screws holding the base from sides of the switch body. Included are a set of 9 self-adhesive Non-Skid feet. When using the Jumbo Switch without the base only 3 feet are needed. (See Photo for proper placement). If using with the base all 9 feet are needed. (See Photo).

Troubleshooting:

Problem: The Jumbo Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Action #4: Make sure the batteries are fresh, this switch will not work without batteries.

Care of Unit:

The Jumbo Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.