

---

---

## **Pull and Play Switch #8042**

---

---

### **USER'S GUIDE**



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### Two Interchangeable grips!

Mount this switch to tabletop, wheelchair tray or bedrail for easy access. It encourages reaching, swiping and grasping skills. It comes with 2 interchangeable grips: 4 inch ball with holes and 3 inch T grip. C-Clamp included. Size: 21"H x 7"W. Weight: 1¾ lbs.

### Operation:

1. The Pull and Play Switch does **not** require any batteries to operate. The switch is table top or wheelchair mountable with the included C-Clamp. When mounting the switch take care not to over tighten the C-Clamp, as this can cause damage to the surface you are mounting to and also the Switch base.
2. Plug the Pull and Play Switch into the jack on your switch-adapted toy/device. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
3. Make sure that the connection between the Pull and Play Switch and your toy/device is tight. There should be no gaps.
4. Chose one of the three included interchangeable grips. Once a grip has been chosen, connect it to the chain via the spring loaded clip.
5. Simply pull on the switch's grip, to activate your toy/device.
6. The toy/device will only remain activated while the Pull and Play Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### Troubleshooting:

**Problem:** The Pull and Play Switch does not activate your toy/device.

**Action #1:** Make sure that the Pull and Play is on a flat surface (not tilted or horizontal). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the Pull and Play Switch and your toy/device is tight. There should be **no** gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #4:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### Care of Unit:

The Pull and Play Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.