

---

---

## Petite Pillow Switch #831-R

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### Decorative patterns!

Our soft, pliable Petite Pillow Switch is ideal for people with limited head and limb movements. The unique switch is encased in foam and covered with a soft bag which can be easily removed and washed. Size: 3"D x 2"H. Weight: 3 oz.

### Operation:

1. The Petite Pillow switch does *not* require any batteries to operate.
2. Plug the Petite Pillow switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Petite Pillow switch and your toy/device is tight. There should be no gaps.
6. Simply press anywhere on the Petite Pillow switch to activate your toy/device.
7. The toy/device will only remain activated while the Petite Pillow switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### Troubleshooting:

**Problem:** The Petite Pillow switch does not activate your toy/device.

**Action #1:** Make sure that the connection between the Petite Pillow switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

**Action #2:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #3:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### Care of Unit:

The Petite Pillow switch's soft cover can be washed.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.