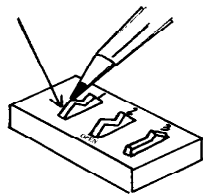

Textured Orbit Ball Switch

Direct from outer space!

Touch this mystery ball and you will be rewarded with glowing rainbow lights while music plays and subtle vibrations are felt! A hidden control on the bottom of your switch allows you to turn any of the rewards on or off. Also can be used as a switch alone. Size: 5½"D x 4½"H. Requires 2 AA Batteries. Weight: 1 lb.

Operation:

1. The Sensi Switch requires 2 AA batteries. The battery compartment is located under the base of the unit. Carefully turn the unit over, and then remove the Philips head screw from the battery cover. Install new batteries, being careful to observe proper (+) & (-) battery polarity. Alkaline batteries are recommended because they last longer. Replace cover and screw.
2. Move the Orbit ball and you will be rewarded with lights, music and vibration. Release pressure from the ball and the Lights, Music and Vibration will stop.
3. To turn off the Lights, Music or Vibration, push the small white rocker switches located on the bottom of the unit, to OFF (Open). See figure below.



4. To use the Orbit Ball as a switch plug one end of the supplied jack cord into the Orbit Ball's jack and the other end into your toy/device. Move the Orbit ball and your toy/device will activate as long as pressure is sustained on the ball. **Please Note:** the Orbit Ball's Lights, Music, and Vibration will also be activated when it is used as a switch, unless they are turned off, via the three way switch.

Troubleshooting:

Problem: The Orbit Ball Switch has no light, music or vibration.

Action: Check the batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: Orbit Ball Switch does not activate a toy or device.

Action # 1: Make sure all connections are tight.

Action # 2: Check batteries in toy/device, replace if weak or dead.

Action # 3: If adapters are being used, they must be Mono only, stereo adapters will not work.

Care of Unit:

The Orbit Ball Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.

**Textured Orbit Ball Switch
#887
with Lights, Music and Vibration**

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com