# Puppy Dog #889

## **USER'S GUIDE**



For Technical Support: Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697 customer support@enablingdevices.com



50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697 www.enablingdevices.com

#### See pooch walk!

Puppy Dog walks, swings his head, barks and wags his tail when switch is activated. A great cause and effect experience. Size:  $5^{"L} \times 6^{"H}$ . Requires Capability Switch and 2 AA Batteries. Weight:  $\frac{1}{2}$  lb.

#### **Operation:**

- 1. Puppy Dog requires 2 AA batteries (not included) to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
- 2. Turn Puppy Dog upside down to reveal the battery compartment. Install 2 AA batteries paying close attention to proper battery polarity.
- 3. Replace the battery compartment cover and secure it. Connect Puppy Dog to any external capability switch (not included) via the jack located at the end of Puppy Dog's cord. Puppy Dog is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to make a tight connection. There should be no gaps.
- 4. Actuate your external capability switch and Puppy Dog will spring to life! He will walk forward and bark.
- 5. Puppy Dog will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Puppy Dog will turn off.

#### Troubleshooting:

Problem: Puppy Dog does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

### Care of Unit:

- Do not inhibit Puppy Dog's movement, as this may cause irreversible damage.
- Do not lift Puppy Dog up by his head, tail or legs, or drop Puppy Dog.
- If you will not be using Puppy Dog for an extended amount of time, we recommend removing the batteries.

**Do not submerge** the unit in water, as it will damage the electrical components. Puppy Dog is surface washable only.