
Push Down Scissors #9074

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Easy to use!

Simply push down on these scissors to easily cut through paper. A spring automatically reopens the scissors once the pressure is released. Great for people with poor hand strength or who are unable to use loop-handled scissors. Comes on a mold with non-skid pads for added stability. *Size: 8¼"L x 4¼"W x 2½"H. Weight: ½ lb.*

PLEASE NOTE: Scissors are sharp and require adult supervision.

Operation:

1. First remove the scissor storage safety guard from the front of the Scissors by applying slight pressure to the push down bar with one hand and then with your other hand pull the guard off the scissors.
2. Use the ramp base to help guide your paper up to the scissors. Next simply push down on these scissors to easily cut through paper. A spring automatically reopens the scissors once the pressure is released.
3. The base of the Push Down Scissors has non skid pads to help hold the scissors in place. Do not place the scissors on top of paper or something that is movable. They should be placed directly on the table or desk surface to take full advantage of the non skid base. When you are finished using the scissors replace guard for storage.

Care of Unit:

The Push Down scissors can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.