

---

---

## Twinkles to Go Octo #9213

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### **Projects magical light!**

This fun night light projects a fun pattern of stars and fish. Choose between two light colors: green or blue. We've adapted the light in two ways: activate one switch and it turns the light on or off; activate the second switch to choose your color pattern. Light has an auto shut off after 45 minutes. Works with long-lasting LED's. Use with or without switches. Size: 4½"L x 4½"W x 4½"H. Requires 3 AAA Batteries. Weight: ¼ lb.

#### **Operation:**

1. Twinkles to Go Octo requires 3 AAA Batteries to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. To install the batteries turn Twinkles to Go Octo over and locate the small screw on the battery cover door. Using a Phillips head screwdriver open the compartment and install 3 AAA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and secure with the screw.
3. Next plug one or two external capability switches (not included) via the jacks located at the side of Twinkles to Go Octo. Twinkles to Go Octo is equipped with two 1/8" female mono jacks for use with external capability switches that have a 1/8" mono plug. Make sure your switches are plugged in all the way. There should be no gaps.
4. One switch jack will replicate pressing on the Twinkles to Go Octo's ON/OFF button and the second switch jack will replicate the color changing button. The Twinkles to Go Octo will also work without switches by activating the locations mentioned earlier. Auto shutoff after 45 minutes if left ON.

#### **Troubleshooting:**

**Problem:** Twinkles to Go Octo does not function correctly.

**Action #1:** Make sure that the connection between the external capability switch or switches and the jacks are plugged in all the way. There should be NO gaps.

**Action #2:** Try your switch with another switch operated device in an effort to rule out your switch as the source of the problem.

**Action #3:** Replace the batteries paying close attention to proper battery polarity.

#### **Care of Unit:**

If you will not be using Twinkles to Go Octo for an extended amount of time, we recommend removing the batteries.

Twinkles to Go Octo can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant.

**Do not use abrasive cleaners, as they will scratch the surface of the unit.**

**Do not submerge the unit, as it will damage the electrical components.**