



See & Speak Object Communicator #9259, Set of 3 #9260

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Contain it!

Do your instructional teaching materials get knocked to the ground? Our large clear container solves that problem. A simple communicator with a large plate to activate your messages about chosen objects. This classroom device provides great auditory and visual cues for your teaching time. Communicator allows you to record four messages on 12 levels with a total recording time of 300 seconds or 6 seconds per message. It also acts as a switch to activate another device further enhancing your lesson. Velcro included for attaching multiple communicators. Size: 10"L x 6½"W x 7½"H. Requires 4 AA Batteries. Weight: 1½ lbs.

Operation:

1. Gently turn the unit over to reveal the black battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and slide the battery compartment cover off. Observing polarity, install four AA size batteries into the holder. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Replace the battery compartment cover and replace the screw.
2. Turn the unit on by turning the ON/OFF/VOLUME knob located on the back of the unit. You will hear an audible click when the unit turns on.
3. To record, simply press and hold the "RECORD" button on the back panel of the unit first, then press down the plate switch. While continuing to hold the record button and the plate switch, speak your message. Your mouth should be about 6 inches from the microphone when recording. Release both the record button and the plate switch when you finish recording.
4. To play back your message, simply activate the plate switch. It is not necessary to continue holding the plate switch when listening to the message. The unit will continue to playback the entire message, even if pressure on the plate is released.
5. The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed.
6. The level knob will allow you to select from one to twelve levels. The message knob is used to select 1 of 4 messages per level (six seconds per message). When you record four messages they will be saved on the level selected, until you re-record over them.
7. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the bottom panel of the unit.

To use as a switch:

8. The See & Speak also works as a switch. It will play your pre-recorded message, as well as activate a switch adapted toy or device. To do this plug one end of the supplied double ended 1/8" to 1/8" cords into the jack on the back of the See & Speak labeled **Device**, and the other end into your toy or device. Press the switch plate and it will play your message and activate the toy or device for as long as the plate is depressed. Your message will only play once even though the connected device keeps activating. You will have to release the switch plate and press it down again to play your message.

Warning: Do not put liquids inside the clear container, doing so will damage the electronics and void warranty.

Important Note:

The communicator allows you to record four messages on 12 levels with a total recording time of 300 seconds or 6 seconds per message.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries paying careful attention to proper polarity.

Action #2: Make sure that you have the unit turned on and the volume turned up to an audible volume.

Action #3: Make sure that you are holding both the record button AND the plate switch SIMULTANEOUSLY while you are recording.

Action #4: Make sure that there is nothing covering the Mic

Problem: The unit is playing old messages fine, but will not record new messages.

Action #1: Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

Care of the Unit:

The See & Speak can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.