

---

---

## ShihTzu Puppy #9301

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### *Lively and sweet!*

This adorable toy dog walks forward, backward and barks when you activate your capability switch. Size: 4 ½ "W x 6"L x 6"H. Weight: ½ lb. Requires 2 AA Batteries.

#### **Operation:**

1. ShihTzu Puppy requires 2 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn ShihTzu Puppy upside down to reveal the battery compartment. Remove the small screw with a Phillips Head screwdriver. Install 2 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and screw.
3. Connect ShihTzu Puppy to any external capability switch (not included) via the jack located at the end of ShihTzu Puppy's cord. ShihTzu Puppy is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to plug your switch all the way in. There should be no gaps.
4. Actuate your external capability switch and ShihTzu Puppy will spring to life! He will walk forward, backward and bark.
5. ShihTzu Puppy will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then ShihTzu Puppy will turn off.

#### **Troubleshooting:**

**Problem:** ShihTzu Puppy does not function correctly.

**Action #1:** Replace the batteries paying close attention to proper (+) & (-) battery polarity.

**Action #2:** Make sure that the connection between the external capability switch and the jack is plugged in all the way. There should be NO gaps.

**Action #3:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

#### **Care of Unit:**

- Do not inhibit ShihTzu Puppy's movement, as this may cause irreversible damage.
- Do not lift ShihTzu Puppy up by his head, tail or legs, or drop ShihTzu Puppy.
- If you will not be using ShihTzu Puppy for an extended amount of time, we recommend removing the batteries.

**Do not submerge** the unit in water, as it will damage the electrical components. ShihTzu Puppy is surface washable only.