
Puppy Gabriel #9321

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Great present for Hanukkah!

Adorable switch-adapted Wilbur sings “Head, Shoulders, Knees and Toes” while touching his toes! Head-To-Toe Wilbur promotes exercise and interaction while teaching parts of the body. His soft plush covering and bright pink color make him an inviting workout buddy. Requires a capability switch (not included). Size: 6"L x 10"W x 11"H. Requires 3 AA batteries. Weight: 1 lb.

Operation:

1. Gabriel Puppy requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is on the bottom of Gabriel Puppy inside the zipper enclosed compartment. Open the flap then, using a Phillips head screwdriver open and remove the batter cover. Next install 3 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it with the screw. Set the ON/OFF switch to ON.
4. Connect Gabriel Puppy to any external capability switch (not included) via the jack located at the end of his cord. Gabriel Puppy is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to plug your switch all the way in. There should be no gaps.
5. Actuate your external capability switch or squeeze Gabriel Puppy’s paw, Gabriel bobs his head, flaps his ears and sings “The Dreidel Song”. Pressing your switch or his paw while he is singing will stop him. Pressing your switch or squeezing his paw again will start him from the beginning of the song.

6. When finished using Gabriel Puppy turn his ON/OFF switch to off to prevent false activations and to prevent battery drain.

Troubleshooting:

Problem: Gabriel Puppy does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is pushed in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Gabriel Puppy for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Gabriel Puppy is surface washable only.