

---

## Anti-contamination Filters #977

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

Our **Sub-Micronic In-Line Filter** prevents infection. It is a hydrophobic filter which will not pass water and has a pore size of .2 PTFE. When used with any Sip and Puff Switch, this unique filter will trap saliva, germs and mucous.

#### INSTALLING AN ANTI-CONTAMINATION FILTER:

1. If your Sip and Puff Switch already has a filter installed in it, simply remove that filter by cutting the tubing on either side OR, more preferably, by twisting it loose and removing it from the tubing. Install the new filter in the same location.
2. If your Sip and Puff does not have a filter already installed, locate the white plastic connector on the line of your Sip & Puff Switch. The #960 does not have a connector. All other models, the connector will be near the middle of the tubing.
3. Carefully remove the connector by cutting the vinyl tubing as shown in *Figure 1*. With #960, simply cut the tubing somewhere in the middle to prepare a location for the filter.

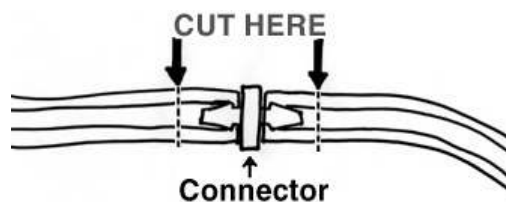


Figure 1

4. Discard used connector.
5. Install the **Anti-Contamination Filter** as shown in *Figure 2*.

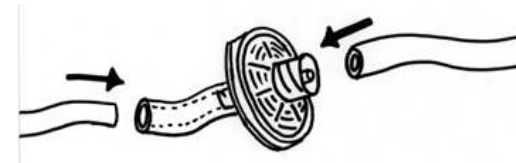


Figure 2

6. Test the Sip & Puff Switch with the new filter to ensure proper installation.

#### IMPORTANT NOTES:

1. Replace the filter when the unit becomes clogged.