

8 Different Scan Modes!

Advanced computer chip technology allows you to choose one of 8 different scanning modes to meet the needs of the user. In modes 1-6, a signal tone sounds as each light illuminates, with a different tone sounding to indicate the end of a sequence—ideal for visually-impaired users. What Scans? A light under each picture turns on sequentially to indicate the message. User can also opt to turn on a beep to accompany the light as the unit scans.

Mode 1 Touch & hold external switch to scan. Release to play message.

Mode 2 Touch & release external switch to scan. Touch external switch again to play message.

Mode 3 Touch & hold external switch #1 to scan; release to stop. Touch external switch #2 to play.

Mode 4 Touch external switch #1 to scan; press again to stop. Touch external switch #2 to play.

Mode 5 Manual scan: Touch external switch #1 to advance scan indicator light one square at a time. Touch external switch #2 to play message.

Mode 6 Manual scan: Touch external switch #1 to advance indicator light one square at a time. A pause in manual activation (adjustable from 1-30 seconds) will play the message.

Mode 7 Auditory Cueing: Touch and hold external switch to allow a portion of each message to play. When desired message is reached, release the external switch to hear the entire message. Has one volume control for the entire message.

Mode 8 Auditory Cueing: Touch and release external switch to begin scan. A portion of each message plays until the external switch is touched again, allowing the full message to play. Has one volume control for the entire message.