



Tube Tracker #5061

USER'S GUIDE

enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Have tons of fun while promoting eye tracking and developing switch access skills. The brightly colored balls move upward on a cushion of air when you activate and hold your switch. Create hundreds of different games. For example, put marks on the tube at various heights and give points for reaching each mark. Includes 4 colored ping pong balls. Size: 7 ¾ "L x 7 ¾ "W x 34"H. Requires 2 AA Batteries and included AC wall transformer included. Weight: 2 ¼ lbs.

Operation:

1. Tube Tracker requires 2 AA batteries (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located on the bottom of the base unit. To remove the battery compartment cover, unscrew the small Phillips Head screw from the compartment, then slide the covers back and off. Install new batteries, being careful to observe proper (+) & (-) battery polarity. Replace cover and screw.
3. Plug the Tube Tracker's AC wall transformer into a 120 Volt wall outlet. Next connect the Tube Tracker to any external capability switch via the 1/8" jack, also located on the side of the unit. If you need to use a ¼" to 1/8" adapter for your switch, it must be a mono adapter, not stereo like our item #1170.
4. Make sure that the connections between the external capability switch and the jack on the Tube Tracker is plugged in all the way. There should be no gaps.
5. Load only one or two supplied balls into the Tube Tracker at once (See photos on back of guide). The top Vinyl cap on the

tube is removable. Replace the Vinyl cap when finished loading the balls. Begin play by activating your external capability switch. The brightly colored balls move upward on a cushion of air when you activate and hold your switch. Once you release your switch the balls will fall back down the tube.

Troubleshooting:

Problem: The Tube Tracker does not work.

Action #1: Do not put more than two balls at a time in the Tube Tracker.

Action #2: Check or replace the 2 AA batteries with fresh new batteries. Never mix different brands or old and new batteries.

Action #3: Make sure the AC wall outlet you are using is working. Try another outlet you know works if not sure.

Action #4: Make sure the Tube Tracker is on a flat level surface and that nothing is restricting any of the balls. Use only the balls included with Tube Tracker, other balls will not work with this product.

Action #5: Try a different switch that you know works with other toys/devices to rule out the Tube Tracker as the source of the problem.

Care of Unit:

The Tube Tracker can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.