

enabling

For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com

Rainbow Switches #7220 (Blue), 7221 (Green), 7222 (Orange), 7223 (Red) (Set of 4) #7224

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Super light touch switches will be your favorite! The slightest touch on these colorful, ergonomic switches will activate any adapted device. Four colors to choose from or purchase all four and save. Size: 5"L x 31/4"W x 11/2"H. Weight: 1/4 lb.

Operation:

- 1. The Rainbow Switch does *not* require any batteries to operate.
- 2. Plug the Rainbow Switch into the jack on your switch-adapted toy/device.
- 3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
- 4. Make sure that the connection between the Rainbow Switch and your toy/device is plugged in all the way. There should be no gaps.
- 5. Simply press anywhere on the Rainbow Switch's colored top to activate your toy/device.
- 6. The toy/device will only remain activated while the Rainbow Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Rainbow Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Rainbow Switch and your toy/device is plugged in all the way. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Rainbow Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a nontoxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.