

Compact Stylus - #9076& Replacement Tips #9084



USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

We designed it for more precise control than longer styluses. A sturdy base and adjustable Velcro strap allow the user to refine its placement – on the palm or back of the hand or anywhere on the arm. Replacement tips sold separately. Swiping and selecting is so easy with our shorter, soft-tipped stylus. Size: 2"L x 2"W x 5"H Weight: 2 oz.

Operation:

1. The Compact Stylus comes with an adjustable Velcro strap, the Compact Stylus can be positioned between any finger, back of the hand or wrist. Once you have the position set you can now secure the Compact Stylus with the Velcro.



2. The Compact Stylus is a flexible rod you can adjust by bending to fit your needs.

Replacement Tips:

When it comes time to replace the Stylus tip you will first need to remove band of silver tape wrapped around the base of the Stylus. Next you will unscrew the tip and replace it with one of the replacement tips (sold separately). The replacement tips will include a new tape strip to re-apply to secure the tip to the Stylus. **Please Note:** The tape must be used to prevent the tip from becoming a choking hazard for young children.

Care of Unit:

The Compact Stylus can be wiped clean with any mild household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.