
Adapted Pie Face #9412

User's Guide



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Adapted Pie Face This exciting and hilarious game is now adapted! Place some delicious whipped cream on the “hand” and activate your switch. The hand could splat at any time. You score a point each time you press your switch without getting pie-faced. Take a risk and hope for the best. Size: 9"L x 8"W x 16"H. Requires 2 AA Batteries. Weight: 2 lbs.

Operation:

1. Adapted Pie Face requires two AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Gently turn the unit over to reveal the black battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and slide the cover off. Observing proper (+) & (-) battery polarity, install batteries into the holder. Replace compartment cover and screw.
3. Connect the Adapted Pie Face to any external capability switch via the jack, located on the base of the unit. Make sure that the connections between the external capability switch and the jack on the Adapted Pie Face is plugged in all the way. There should be no gaps.
4. The Adapted Pie Face is equipped with a 1/8-inch jack, for use with capability switches with a 1/8-inch plug. If you need to use a 1/4- to 1/8-inch adapter use our No.1170 adapter. If purchased elsewhere, it must be a mono adapter, not stereo.
5. Now actuate your external capability switch Place some delicious whipped cream on the “hand” (or use the included sponge) and activate your switch. The hand could splat at any time. You score a point each time you press your switch without getting pie-faced. Take a risk and hope for the best.

6. The Adapted Pie Face will only continue rotating the handle while the external capability switch is actuated. Once you release the switch, the Adapted Pie Face will stop.

Please Note: Review the included original manufactures setup and game play instructions.

Important Note:

The Adapted Pie Face can also be used normally without a switch by turning the one handle. Each rotation of the handle brings you one step closer to getting hit.

Troubleshooting:

Problem: The Adapted Pie Face does not rotate the handle when your switch is pressed.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Action#3: Make sure that you have plugged your switch in all the way. There should be no gaps.

Action#4: Try your external capability switch with another toy/device to rule out the switch as the source of the problem.

Care of the Unit:

The Adapted Pie Face can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non- toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.

Rev 11/13/17