
Rusty #1074

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Our adorable painted pony will put a smile on everyone's face. Just activate your capability switch and Rusty trots to the lively theme song from The Lone Ranger ("William Tell Overture"). Great for teaching cause and effect and for developing tracking. Size: 11"L x 4"W x 8"H. Requires 3 AA Batteries. Weight: ¾ lb.

Operation:

1. Rusty requires 3 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn Rusty over and open up the Velcro enclosure, then using a Phillips head screwdriver remove the screw and lift the battery cover off. Observing proper (+) & (-) battery polarity, install 3 AA size batteries into the holder. Replace the battery compartment cover and screw.
3. Set the ON/OFF switch to the ON position. **Please Note:** Due to switch adaption Rusty will only work through the use of an external switch. The built in musical note button on his side will not work.
4. Connect Rusty to any external capability switch via the cord input jack. Make sure all connections are pushed in all the way, there should be no gaps. Press and hold your capability switch and Rusty trots to the lively theme song from The Lone Ranger ("William Tell Overture"). Great for teaching cause and effect and for developing tracking.
5. One press of your switch will start Rusty moving, while playing the theme song from The Lone Ranger ("William Tell Overture"). A second press of your switch will stop him. If you hit your switch again he will start back up from the beginning of the song.

6. Turn OFF Rusty when finished using to prevent false activation and battery drain.

Troubleshooting:

Problem: Rusty does not work.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Action #3: Make sure that you have plugged your switch in all the way. There should be no gaps.

Action #4: Try your external capability switches with another device to rule out the switches as the source of the problem.

Care of the Unit:

If you will not be using Rusty for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Rusty is surface washable only.