
Theo & Rio #9329

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

A perfect pair of friends! Theo is a brown bear that sings “Lovely Day” and Rio is the gray squirrel that likes to dance along. Great for stimulating visual and auditory attention. Works with or without a switch. Size: 8"L x 9"W x 11½" Weight: 1¼ lbs. Battery: 3 AA

Operation:

1. Theo & Rio requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located on the bottom of Theo & Rio. Open the Velcro flap, then using a Phillips head screwdriver open and remove the batter cover. Next install 3 AA batteries paying close attention to proper battery polarity.
3. Set the On/Off switch to On. Replace the battery compartment cover and secure it with the screw.
4. Connect Theo & Rio to any external capability switch (not included) via the jack located at the end of his cord. Theo & Rio is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have pushed the switch plug all the way in. There should be no gaps.
5. Actuate your external capability switch or squeeze Theo & Rio's foot with the musical note symbol and he will sing the song “Lovely Day”. Great for stimulating visual and auditory attention
6. Each switch activation or foot squeeze will start or stop Theo & Rio. If you hit the switch or squeeze his arm again during play, the music will start from the beginning.

Troubleshooting:

Problem: Theo & Rio does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch is pushed in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Theo & Rio for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Theo & Rio is surface washable only.