
Rock & Roll Rider #9336

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Liven up your holidays with this rockin' reindeer! Decked out in a festive hat and scarf, this adorable plush toy trots and shakes its tail to "Sleigh Ride" when you activate your switch.

Size: 8"L x 7"W x 10"H. Requires 3 AA Batteries. Weight: 1½ lbs.

Operation:

1. Rock & Roll Rider requires 3 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn Rock & Roll Rider over and open up the Velcro enclosure, then using a Phillips head screwdriver remove the screw and lift the battery cover off. Observing proper (+) & (-) battery polarity, install 3 AA size batteries into the holder. Replace the battery compartment cover and screw.
3. Set the ON/OFF switch to the ON position. **Please Note:** Rock & Roll Rider will work through the use of an external switch or the built in musical note button on his side.
4. Connect Rock & Roll Rider to any external capability switch via the cord input jack. Make sure your switch is plugged in all the way, there should be no gaps. Press and release your capability switch and Rock & Roll Rider trots and shakes its tail to "Sleigh Ride". Great for teaching cause and effect and for developing tracking. If you press your switch while he is playing Rock & Roll Rider will stop. Pressing and releasing your switch again will start the song from the beginning.
5. Turn OFF Rock & Roll Rider when finished using to prevent false activation and battery drain.

Troubleshooting:

Problem: Rock & Roll Rider does not work.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Action #3: Make sure that you have plugged your switch in all the way. There should be no gaps.

Action #4: Try your external capability switches with another device to rule out the switches as the source of the problem.

Care of the Unit:

If you will not be using Rock & Roll Rider for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Rock & Roll Rider is surface washable only.