
Smooth Moves Sloth #7579

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

A new best friend!

Lights, sounds and songs play while Smooth Moves Sloth's head moves side to side. Just activate your capability switches to get the fun started. 85 songs, and phrases help introduce counting and colors. Size: 6"L x 5"W x 9"H. Requires 4 AA Batteries. Weight: 1½ lbs.

Operation:

1. Smooth Moves Sloth requires 4 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types of batteries together.
2. The battery compartment is located under the toy's. Carefully remove the screw from the cover using a Phillips head screwdriver, and lift the cover off. Observing proper (+) & (-) battery polarity, install 4 AA size batteries into the holder. Replace the cover and screw. Next set the ON/OFF/VOLUME switch to Low or High.
3. Smooth Moves Sloth has two external switch input cord jacks so you can use your own capability switches if needed. To use an external switch, plug your switch into the desired jack on either side of Smooth Moves Sloth. Make sure your switch is plugged in all the way. There should be no gaps. Activating your switch will replicate the corresponding built in button on the front of Smooth Moves Sloth.
4. Just activate your capability switches or the two buttons on the front of Smooth Moves Sloth to get the fun started. Lights, sounds and songs play while Smooth Moves Sloth's head moves from side to side. 85 songs and phrases help introduce counting and colors.

For more detailed information on all Smooth Moves Sloth feathers and functions please read the original manufactures user's guide.

Troubleshooting:

Problem: Smooth Moves Sloth does not work.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Problem: Smooth Moves Sloth does not work with a switch.

Action: Try a different switch with Smooth Moves Sloth to rule out that Smooth Moves Sloth is the source of the problem.

Care of the Unit:

Smooth Moves Sloth can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.