
Narwhal Bubble Blower & Rainbow Switch Bundle #2336

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697

www.enablingdevices.com

Six spinning bubble wands make for a whale of a good time! This little guy can spout bubbles all day long. Just fill his mouth with bubble juice, press the included rainbow switch and a stream of bubbles comes floating out! Comes with 4 oz. of non-toxic, dye-free bubble juice. Includes our popular light touch Rainbow switch (color will vary) Size: 4¾"L x 3½"W x 4½"H. Requires 3 AA batteries. Weight: ½ lb.

Operation:

1. Narwhal Bubble Blower requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located under bubble blower's base. Carefully turn the bubble blower over to reveal the battery compartment cover. Using a Phillips head screwdriver unscrew the cover screw and install 3 AA batteries, paying close attention to proper (+) and (-) battery polarity.
3. Replace the battery compartment cover and secure with the screw. Connect to any external capability switch (included) via the jack located on the side of the bubble blower. Narwhal Blower is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have your switch plugged in all the way. There should be no gaps.
4. Narwhal Bubble Blower can be used with or without an external switch. The ON/OFF switch is located in the back of the bubble blower. When using an external capability switch, the ON/OFF switch needs to be set in the OFF position. Actuate your external capability switch and the bubbles will start to flow! Once you release your switch the bubbles will stop.
5. When finished playing with Narwhal bubble blower turn the bubble blower OFF to prevent battery drain.

Troubleshooting:

Problem: Narwhal Bubble Blower does not function correctly.

Action #1: Check that all the batteries are installed, following proper (+) & (-) polarity. Also check that all batteries are fresh.

Action #2: Replace the batteries paying close attention to proper (+) and (-) battery polarity.

Action #3: Make sure you have enough Bubble solution in the dipping troth. The Bubble solution should cover the Bubble Dippers completely.

Action #4: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #5: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Narwhal Bubble Blower for an extended amount of time, we recommend removing the batteries.

Narwhal Bubble Blower can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.