German Shepherd Puppy #63

USER'S GUIDE



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697

www.enablingdevices.com

Bow Wow!

Take your German Shepherd Puppy for a walk. He moves and wags his tail while barking. He is a wonderful pet to have, so soft and cuddly, and helps to increase directionality skills. Size: 7½"L x 3¾"W x 7"H. Requires Capability Switch and 2 AA Batteries. Weight: 1 lb.

Operation:

- German Shepherd Puppy requires 2 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
- 2. Turn German Shepherd Puppy upside down to reveal the battery compartment. Remove the small screw with a Phillips Head screwdriver. Install 2 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and screw.
- 3. Connect Puppy to any external capability switch (not included) via the jack located at the end of Puppy's cord. German Shepherd Puppy is equipped with a 1/8" jack for use with external capability switches with a 1/8"plug. Make sure to make a tight connection. There should be no gaps.
- 4. Actuate your external capability switch and St. Bernard Puppy will spring to life! He walks forward, backwards, barks and wags his tail.
- 5. German Shepherd Puppy will remain on while the external capability switch continues to be actuated. Once you release the external capability switch Puppy will turn off.

Troubleshooting:

Problem: German Shepherd Puppy does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) & (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit German Shepherd Puppy's movement, as this may cause irreversible damage.
- Do not lift German Shepherd Puppy up by his head, tail or legs, or drop German Shepherd Puppy.
- If you will not be using German Shepherd Puppy for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Puppy is surface washable only.