
Mini Puppy Dog #889M

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

See pooch walk!

This adorable, lifelike little pup barks, wags his tail and swings his head when his human playmate activates an external capability switch (not included).. A great cause and effect experience. Size: 5"L x 6"H. Requires Capability Switch and 2 AA Batteries. Weight: ½ lb.

Operation:

1. Mini Puppy Dog requires 2 AA batteries (not included) to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Mini Puppy Dog upside down to reveal the battery compartment. Install 2 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it. Connect Puppy Dog to any external capability switch (not included) via the jack located at the end of Puppy Dog's cord. Puppy Dog is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to make a tight connection. There should be no gaps.
4. Actuate your external capability switch and Mini Puppy Dog will spring to life! He will walk forward and bark.
5. Mini Puppy Dog will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Puppy Dog will turn off.

Troubleshooting:

Problem: Mini Puppy Dog does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Mini Puppy Dog's movement, as this may cause irreversible damage.
- Do not lift Mini Puppy Dog up by his head, tail or legs, or drop Puppy Dog.
- If you will not be using Mini Puppy Dog for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Mini Puppy Dog is surface washable only.