
Let Loose Lenny #9344

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Lenny will make you want to shout with joy! He boogies to the song "Shout" when you press his foot or your external capability switch (not included). You'll want to sing and dance with this talented king of the jungle. His wild mane is soft and can be tamed with a small comb or soft brush for extra sensory input. Lenny sings for 45 seconds or until you press the switch again. Size: 9"L x 8"W x 13½"H. Requires 3 AA Batteries. Weight: 1 lb.

Operation:

1. Lenny requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartments are located on the bottom of Lenny, feet. Using a Phillips head screwdriver open and remove the batter covers. Next install 3 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment covers and secure them with the screws. Set the ON/OFF switch to ON.
4. Connect Lenny to any external capability switch (not included) via the jack located at the end of his cord. Lenny is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have pushed the switch plug all the way in. There should be no gaps.
5. Actuate your external capability switch or squeeze Lenny's Musical Note (foot) and he boogies to the song "Shout" when you press his foot or your external capability switch (not included)

Troubleshooting:

Problem: Lenny does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch is pushed in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Lenny for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Lenny is surface washable only.