
Pink Baby Poodle #936

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Woof, Woof!

Take your baby poodle for a walk by activating your capability switch and it will move about forward and backwards raising its head and making cute barking sounds. Size: 5½"L x 6 ½ x "W x 3"H. Requires 2 AA Batteries. Requires 2 AA Batteries. Weight: ¾ lb.

Operation:

1. Pink Baby Poodle requires 2 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Pink Baby Poodle upside down to reveal the battery compartment. Remove the small screw with a Phillips Head screwdriver. Install 2 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and screw.
3. Connect Pink Baby Poodle to any external capability switch (not included) via the jack located at the end of Pink Baby Poodle's cord. Pink Baby Poodle is equipped with a 1/8" jack for use with external capability switches with a 1/8"plug. Make sure to have a tight connection. There should be no gaps.
4. Actuate your external capability switch and Pink Baby Poodle will spring to life! She will walk forward; raise her head, bark and then backup.
5. Pink Baby Poodle will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Pink Baby Poodle will turn off.

Troubleshooting:

Problem: Pink Baby Poodle does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) & (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is plugged in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Pink Baby Poodle's movement, as this may cause irreversible damage.
- Do not lift Pink Baby Poodle up by her head, tail or legs, or drop Pink Baby Poodle.
- If you will not be using Pink Baby Poodle for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Pink Baby Poodle is surface washable only.