

**Action #6:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

**Care of Unit:**

If you will not be using the Bubble Machine for an extended amount of time, we recommend removing the batteries.

The Bubble Machine can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

---

## CoComelon Musical Bubble Machine #2340

---

---

### USER'S GUIDE



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

You don't have to be a fan on CoComelon to love this bubble Machine! Use sustained pressure on your external capability switch (not included) or the ladybug on/off switch for a continuous stream of bubbles. No-spill reservoir holds 3+ ounces of bubble solution (included). For music, just move the ladybug switch down and you'll hear "The Wheels on the Bus" as bubbles float into the air. (External capability switch does not activate music.) Size: 6½"D x 7¾"H. Requires 3 AA Batteries. Weight: 1 lb.

### **Operation:**

1. CoComelon requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located on the side of the bubble blower. Turn the bubble blower to reveal the battery compartment cover. Using a Phillips head screwdriver unscrew the cover and install 3 AA batteries, paying close attention to proper (+) and (-) battery polarity.
3. Replace the battery compartment cover and secure with the screws. Connect to any external capability switch (not included) via the cord jack located at the back of the Bubble Machine's base. CoComelon Bubble Machine is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have your switch plugged in all the way. There should be no gaps.
4. To fill the Bubble Machine, unscrew the top of the included bubble solution bottle. Next remove the seal on the top of the bottle. Unscrew the solution reservoir on the bottom of CoComelon. Fill with bubble solution. Make sure the tubes go into the opening of the reservoir and are not pinched when screwing it to the base.
5. The Bubble Machine can be used with or without an external

switch. The ON/OFF Ladybug switch is located in the front of the unit. When using an external capability switch, the ON/OFF switch needs to be set in the OFF position. Actuate your external capability switch and the bubbles will start to flow! Once you release your switch the bubbles will stop.

6. When finished playing with Bubble Machine turn OFF to prevent battery drain.

**Please Note:** External capability switch does not activate music with bubbles mode.

### **Troubleshooting:**

**Problem:** The Bubble Machine does not function correctly.

**Action #1:** Check that all the batteries are installed, following proper (+) & (-) polarity. Also check that all batteries are fresh.

**Action #2:** Replace the batteries paying close attention to proper (+) and (-) battery polarity.

**Action #3:** Make sure you have enough Bubble solution in the solution bottle. The Bubble solution should cover the tube enough to syphon the solution.

**Action #4:** Clean the tubes with warm water. Place the tubes in warm water and turn the bubble Machine on. Let the bubble Machine run for 2 or 3 minutes to allow the water to flush out the bubble solution.

**Action #5:** Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

**(continued on the back page of this guide.)**