

Note:

Small children should be supervised when playing with this toy, as there are some small parts, which could be a choking hazard.

Care of Unit:

- Do not inhibit Tabby Kitten's movement, as this may cause irreversible damage.
- Do not lift Tabby Kitten up by her tail, head or legs, or drop Tabby Kitten.
- If you will not be using Tabby Kitten for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components.

Tabby Kitten #9237

USER'S GUIDE



For Technical Support:

**Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m.
(EST)**

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

This little tabby cat meows while it nods its head and wags its tail, then goes for a walk while you sustain pressure on its external capability switch (not included). Keep pressing the switch and it even runs! Plush "fur" is extra soft to the touch. Size: 6"L x 3"W x 6"H. Requires 2 AA Batteries. Weight: $\frac{3}{4}$ lb.

Operation:

1. Tabby Kitten requires 2 AA batteries (Not included) to operate. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well.
2. Turn Tabby Kitten upside down to reveal the battery compartment. To open the compartment use the tip of a pen to depress the release button. Next slid the cover back and then open. Install 2 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover slid it forward, making sure not to pinch any wires. The door will lock closed.

Please Note: Due to switch adaption the ON/OFF switch has been disabled.

4. Connect Tabby Kitten to any external capability switch (not included) via the jack located at the end of the kitten's cord. Tabby Kitten is equipped

with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure plug your switch in all the way. There should be no gaps.

5. Actuate your external capability switch and the Tabby Kitten will spring to life!
6. Tabby Kitten will walk, wag her tail and cry. If you pick her up, she will stop and make a cute "Mew Mew" sound.
7. Tabby Kitten will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Tabby Kitten will turn off.

Troubleshooting:

Problem: Tabby Kitten does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.