

---

---

# Alphabet Jodey #9351

---

---

## USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

Learn letters and numbers with Alphabet Jodey! This little giraffe sways and wiggles their ears while singing "Ten Little Elephants" and "The ABC Song." Press the foot or your external capability switch and Jodey sings until you press the switch again. Each activation changes the tune. Colorful letters and numbers adorn Jodey's bandana. Size: 9"L x 8"W x 10"H. Requires 3 AA Batteries. Weight: 1¼ lb.

### **Operation:**

1. Alphabet Jodey requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located on the bottom of Alphabet Jodey. Open the Velcro flap, then using a Phillips head screwdriver open and remove the batter cover. Next install 3 AA batteries paying close attention to proper battery polarity.
3. Set the On/Off switch to On. Replace the battery compartment cover and secure it with the screw.
4. Connect Alphabet Jodey to any external capability switch (not included) via the jack located at the end of his cord. Alphabet Jodey is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have pushed the switch plug all the way in. There should be no gaps.
5. Actuate your external capability switch or squeeze Alphabet Jodey foot with the musical note symbol. Jodey sings until you press the switch again. Each activation changes the tune.

6. Each activation changes the tune he sings. His mouth moves as he sings and his arms go up and down in time to the music.

### **Troubleshooting:**

**Problem:** Alphabet Jodey does not function correctly.

**Action #1:** Replace the batteries paying close attention to proper battery polarity.

**Action #2:** Make sure that the connection between the external capability switch is pushed in all the way. There should be NO gaps.

**Action #3:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

### **Care of Unit:**

If you will not be using Alphabet Jodey for an extended amount of time, we recommend removing the batteries.

**Do not submerge** the unit in water, as it will damage the electrical components. Alphabet Jodey is surface washable only.