

**Language Selection:**

Press and hold the round language button for 2 seconds to switch between languages. E.T. will cycle through languages. Continue until you hear "Ready" in your preferred language then release the button.

**Day Time Mode:**

This is the default mode when E.T. is switched ON. Press your switch to hear E.T. say 9 different phrases from the classic film and feel his heart.

His heart rate will change depending on his mood, and his glowing heart will beat in time.

To preserve battery life E.T. will automatically go to sleep after approximately 12 seconds without input.

E.T. can be turned on by pressing and releasing your switch or by sliding his ON/OFF switch.

**Night Time Mode:**

Press and hold your switch for 2 seconds to activate NightTime mode. E.T. plays the theme tune lullaby for approximately 10 minutes with a gentle heartbeat, and then he will go to sleep. Pressing and holding your switch for another 2 seconds will stop the song from playing and bring you back to Day mode.

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697

---

---

## My Best Friend E.T. #9352

---

---

### USER'S GUIDE



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

E.T. phone home! This adorable switch-adapted plush says his famous line as well as many others. Press and release your external capability switch (not included) to hear 9 different phrases. Hold the switch down for 2 seconds and E.T. plays the iconic theme from the beloved movie. His heart lights up and gently beats at a soothing 67 beats per minute, mimicking a resting heartbeat. E.T. has 14 language options, including English, French, Spanish and German. On/off switch saves battery life. Includes a stand for easy play. Size: 5½"L x 8"W x 11"H. Requires 3 AAA Batteries. Weight: 1½ lbs.

#### **Operation:**

1. E.T. requires 3 AAA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located inside the Velcro enclosed compartment on his back. Open the flap then, using a Phillips head screwdriver open and remove the batter cover. Next, install 3 AAA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it with the screw. Set the ON/OFF DEMO mode switch to ON. E.T. Will say "Ready". DEMO mode will have limited function. For the full set of instructions please refer to the included original manufactures instruction booklet. We have included a stand to use with E.T. so that you can sit him up when using a switch. There is Velcro on the back of E.T. to hold him in place on the stand. There is also a cutout at the base of the stand to put his switch cable through.  
\*\*\***Please Note:** Due to switch adaption, E.T.'s heart button has been disabled. E.T. will only work with an external switch.
4. Connect E.T. to any external capability switch (not included) via the jack located at the end of his cord, E.T. is equipped with a 1/8" jack for use with external capability switches with a 1/8"

plug. Make sure to plug your switch all the way in. There should be no gaps. See back page of this guide for detailed description of E.T.'s modes.

5. Press and release your external capability switch (not included) to hear 9 different phrases. Hold the switch down for 2 seconds and E.T. plays the iconic theme from the beloved movie. His heart lights up and gently beats at a soothing 67 beats per minute, mimicking a resting heartbeat. E.T. has 14 language options, including English, French, Spanish and German. Pressing and holding your switch for another 2 seconds will stop the song from playing.
6. When finished using disconnect your switch to prevent false activations and to prevent battery drain.

#### **Troubleshooting:**

**Problem:** E.T. does not function correctly.

**Action #1:** Replace the batteries paying close attention to proper battery polarity.

**Action #2:** Make sure that the connection between the external capability switch and the jack is pushed in all the way. There should be NO gaps.

**Action #3:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

#### **Care of Unit:**

If you will not be using E.T. for an extended amount of time, we recommend removing the batteries.

\*\*\***Do not submerge** the unit in water, as it will damage the electrical components. E.T. is surface washable only.