

**AUDITORY CUEING:**

During Message Scanning you can enable the communicator to speak the message being scanned, but at reduced volume. This gives the user an auditory “preview” of the current message. Pressing the capability switch again will play the current message at full volume. To turn Auditory Cueing ON or OFF, us the toggle switch on the back of the unit.

**Icon Inserts:**

The communicator uses clear transparency or standard printer paper to make the icon inserts that you can print from your computer. The icon or picture holder uses 4" x 5" icons. To place your icons or pictures, lift up on the clear tab just enough to slide your icons or pictures in.

**REPLACING BATTERIES:**

Replace batteries regularly. Weak batteries will cause the communicator to act erratically, such as shutting off while playing messages.

Use brand name ALKALINE batteries ONLY (e.g. Duracell or Eveready Energizer).

Do NOT use heavy duty or rechargeable batteries.

Do NOT mix new and old batteries.

Do NOT over tighten the battery compartment screw.

To extend battery life, always shut the communicator OFF when not in use.

Always remove batteries when the unit will not be used for extended periods of time. Dead batteries often leak acid and can cause permanent damage to the communicator.

**PRECAUTIONS:**

Always keep liquids far away from the communicator to prevent permanent damage to internal circuitry from spillage.

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

---

## Inline 4 Comm. For The VI #7089

---

---

## USER'S GUIDE



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

This brand new communicator is designed for the visually impaired but can be beneficial for anyone who needs a larger target area. This in-line communicator has 12 levels with 500 seconds of recording time; a jack for automatic scanning; and large, brightly illuminated targets that make it possible to see, select and activate messages. The target lights up when activated and the pre-recorded message plays. Four- by five-inch icons slip easily under clear overlays. Size: 18"L x 6½"D x 3½"H. Requires 6 AA Batteries. Weight: 3½ lbs.

#### Features:

- 12 Levels with 500 seconds of record time
- Recordable level announcer
- Single jack for automatic scanning w/LED illumination
- True Voice recording quality
- Adjustable scanning speed
- Volume Control
- On/Off Auditory Cueing

#### BATTERY INSTALLATION:

Make sure unit is in the OFF position by rotating the ON-OFF-VOLUME control counter clockwise until a "click" is heard. Remove the screws on battery compartment cover and install 6 brand new "AA" alkaline batteries. Do not use any other type of AA batteries. No Heavy Duty, Super Duty, Rechargeable or Lithium Ion. Replace battery cover and screws.

#### OPERATION:

Rotate the ON-OFF-VOLUME control to approximately its' midway position. The unit is now on Level One of the twelve levels provided in the Communicator. **Please Note:** From the OFF position the communicator will always start on level 1.

#### PLAYING MESSAGES:

At this time, you can access each of the 4 messages on Level One by pressing any one of the four icons, one at a time. Or through the use of an external switch via scanning.

#### RECORDING MESSAGES:

Using one hand, press and hold the Record button on the backside of the unit. Using your other hand, press the icon you wish to record and speak into the microphone. Releasing the icon or Record button will cancel recording. If you are going to record more than one message, you may press and hold the record button while pressing the appropriate icons to record your messages.

#### RECORDING LEVEL ANNOUNCEMENT MESSAGE:

Your Level Announcement Message may be changed at any time and can be renamed "Level One the Food Level" or "Level One Jimmie's Messages", etc. Simply press the Record button first, then press the Level button and speak your Level Announcement Message into the microphone. Note: to hear your new level message you must either cycle through all 12 levels by repeatedly pressing the Level announcement button until it returns to Level 1.

#### CHANGING LEVELS:

Each time the Level button is pressed, the communicator will advance to the next level. Pressing the Level button on Level 12 will return to Level 1. Simply follow the instructions above to record each of the four-communicator messages or your Level Announcement Message for each level. Remember, to play your Level Announcement Message, you **MUST** cycle through all remaining levels and return to that particular level to hear that Level Announcement Message.

#### MESSAGE SCANNING:

The communicator has built-in Automatic Scanning. Plug a capability switch into the 1/8" jack located on the rear of the communicator and press the capability switch. Upon release, the unit will scan from left to right and continue scanning until the next press of the capability switch.

#### SCAN SPEED:

You can control the speed of the Message Scanning by adjusting the scan speed knob. Turning the knob clockwise increases scan speed. Counter-clockwise decreases scan speed.

#### CHANGING THE NUMBER OF MESSAGE FIELDS:

With the unit **ON**. First press the Message Limit button on the back of the unit and keep it depressed while pressing the last message location you want to stop at. The unit will always default to 4 messages after it has been turned **OFF** and turned back **ON**. Please Note: This function is for scanning with an external switch only. This will not affect pressing on the 4 message plates directly.