

---

---

## Crawl N Colors Chameleon #1217

---

---

### USER'S GUIDE

---

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

  
**Enabling Devices**

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

Teach ABCs, 123s, and colors with this colorful chameleon. Adapted for use with one or two switches, it sings more than 25 songs about letters, numbers and colors when activated. The chameleon's tail and microphone light up and flash to the beat of the song. It's a push-along toy, too, for added fun and motor skill practice. Can also be used without a switch. Size: 8¾"L x 6¼"W x 3¾"H. Requires 3 AA Batteries. Weight: 1 lb.

### **Operation:**

1. Gently turn the Roll & Learn Turtle over to reveal the battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and slide the battery compartment cover off. Observing polarity, install 3 AA size batteries into the holder. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Replace the battery compartment cover and replace the screw.
2. Plug any capability switch into either of the 1/8" jacks located on the back of the Chameleon, make sure the switch is plugged in all the way. There should be no gaps between the connections, if there is a gap the Chameleon will stay ON and your switches will have no control over the Chameleon.
2. For one jack. Spin the roller or activate your switch and listen to fun songs. Phrases and music. For jack 2. Press the ABC, 123 Button or activate your switch and Chameleon introduces number, letters and colors through phrases, fun sounds and lights! Encourages discovery and exploration and teaches cause and effect.

**Please Note:** Crawl N Colors Chameleon is part of the Link Squad toys. See the original included instructions on how to connect him to his pals and learn about how they interact with each other.

### **Troubleshooting:**

**Problem:** Crawl N Colors Chameleon fails to operate, or operates incorrectly.

**Action #1:** Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries.

**Action #2:** Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

**Please Note:** When you are finished using Crawl N Colors Chameleon remove your external switches. This will help prevent false activations and conserve battery life. For prolonged storage we recommend removing the batteries to prevent battery drain and or battery leakage which can cause damage Crawl N Colors Chameleon's electronics.

### **Care of the Unit:**

Crawl N Colors Chameleon can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.