

Communication Builder EZ Mount Plate

#1604



enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914-478-0960 / Fax 914-478-7030
www.enablingdevices.com

“A game changer”—that’s what our resident Occupational Therapist calls this new mounting system for our Communication Builders. Originally designed for a customer whose wheelchair was overloaded with traditional mounts, the EZ Mount system is now available to everyone at an affordable price.

How it works: For a fast and easy mounting solution, all you need is the EZ Mounting Plate and a mounting arm for your Communication Builder. For more versatility, the EZ Mount Plate gives you two mounting options:

1. An adapter that integrates with a gooseneck or other mounting arm that has a ¼-20 female threaded socket.
2. An adapter that is compatible with all of our current Magic Arm mounts with a 5/8-inch diameter post.
3. A mounting adapter for tubular rails of 1", 7/8", 3/4", or 5/8" in diameter is also available (sold separately).

OPERATION:

The photos below show the sequence of assembling the mounting plate with the proper adapter to fit your mounting system. Option one is for mounting to any mount with a 5/8" diameter post, such as our Light Duty Magic Arm Mounting System #1557 or our Magic Arm Mounting System #1631. First, attach the mounting adapter with the 5/8" diameter hole and single adjustable thumbscrew to your device’s mounting plate with the two Phillips head screws.





Next you will need to attach the mounting plate to the back of your Communication Builder. Use the included 4 long Philips Head screws. Please note the orientation of the mounting plate. It the screws will only line up one way as shown in the photo.



Next, line up the thumbscrew with the flat spot on the mounting arm's post. You will need to back the thumbscrew almost all the way out to allow the block to seat all the way down on the post. Once this is done, tighten the thumbscrew to secure the mounting plate to your mounting arm.



Option 2 is for mounting to a gooseneck mounting system with a $\frac{1}{4}$ -20 threaded female socket. Attach your device's mounting tray to the adapter and secure with the two Phillips head screws. Thread the adapter with the $\frac{1}{4}$ -20 screw into your gooseneck. Place your device into the mounting tray and press down to lock in place. See photos.





Troubleshooting:

Problem: Mounting Plate is loose on the mounting Adapter.

Action: Make sure that you have secured the mounting Plate to the adapter with the included two short Philips head screws, and that they are tightened properly.

Problem: Mounting adapter is loose on mounting arm.

Action #1: Make sure the thumbscrew, is lined up with the flat spot on your 5/8" mounting post. Second, make sure the thumbscrew is tightened all the way.

Action #2: Make sure you have tightened the 1/4-20 post all the way to your gooseneck.

Care of Unit:

The mounting Plate and adapters can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic, biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

- If you have tried all of the above and are still having problems, contact our Tech Support Department at 1-800-832-8697 ext. 320 or via email at: customer_support@enablingdevices.com

Rev 3/12/2025