
Switch-Adapted Bluetooth Speaker #4098 (PEICHENG Model)

Care & Maintenance:

1. Keep the Adapted Bluetooth Speaker clean by wiping it with a slightly damp cloth when needed. It can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.
2. Keep the Adapted Bluetooth Speaker out of direct sunlight and away from any direct heat source.
3. Do not drop the Adapted Bluetooth Speaker on a hard surface and do not expose it to moisture or immerse it in water.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

This mini Bluetooth speaker packs a lot of sound into a small package. Adapted for use with any of our switches, you can easily control volume up, volume down, play and pause. Pair with any Bluetooth device or use with your prerecorded micro SD card. The included wrist strap makes it easy to carry with you. Three external switches required (Not included); switches do not control the on/off function. Size: 3"L × 3"W × 1½"H. USB charging. Weight: ¼ lb.

Please Note: Computer required for downloading music to Micro/SD/TF card.

Operation:

1. The Adapted Bluetooth Speaker has a built in rechargeable battery, which is charged through your computer via the included USB cable or using the included cable with any Tablet or Smartphone wall charger for faster charging. You can use the Adapted Bluetooth Speaker during charging.
2. To charge Adapted Bluetooth Speaker, plug it into a USB port on your computer while the computer is on or using the USB cable or a better way to charge it is with a Table or Smartphone wall charger. The charging port and SD card slot is located on the bottom of the speaker behind the rubber cap. The LED light will glow red while charging with the power OFF. When charging completes, (approximately 3 to 4 hours) the light will go OFF with the power OFF.

Modes of Play:

The Adapted Bluetooth Speaker can playback music via Bluetooth, MicroSD/TF, Press the ON/OFF switch (short press) to change modes. You will hear an audio prompt for each mode.

Please Note: FM Mode does not function on this device.

Please Reference the original manufacture instructions for all of the Adapted Bluetooth Speaker functions and features and any additional troubleshooting info.

Please Note: Some features or functions listed in the original manufactures instructions may not work due to switch adaption.

3. To listen to music stored on a Micro SD/TF, insert the card into the appropriately marked slot on the bottom of the Speaker. Turn ON Adapted Bluetooth Speaker's ON/OFF switch ON. Connect your switch to the cord jack marked "Play/Pause". Press once (short press) to Play. Press (short press) a second time to pause the music. To move to the Next song, press your switch 2 times. To hear the previous song press your switch 3 times. Connect two additional switches to the marked cords to control Volume Up, & volume Down. To adjust volume, use your switches by pressing and holding them down to raise or lower the volume level.
4. When you are finished using the Adapted Bluetooth Speaker make sure to turn the ON/OFF switch on the side of the speaker to OFF, to prevent the battery from draining down all the way. If not and the battery drains out and left in this state could cause damage to the battery.

Troubleshooting:

Problem: Adapted Bluetooth Speaker does not work.

Action #1: Make sure you have charged the Adapted Bluetooth Speaker. Blue LED ON: Charge, Blue LED OFF: Full Power & Blue LED Flashing: Low Power.

Action #2: Make sure that there is nothing inhibiting the depression of the button switches or holding any of the button switches down.

Action #3: Try reloading your music on the MicroSD/TF card, or try a different MicroSD/TF card to rule this out as the issue.