
Colt #1076

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Yeehaw!

Our adorable painted pony will put a smile on everyone's face. Just activate your capability switch or press his ear and Colt trots ahead and says one of five Western-themed phrases. Press his saddle and he trots and flicks his tail to the lively theme song from The Lone Ranger ("William Tell Overture"). Great for teaching cause and effect and for developing tracking skills. Works with or without an external switch (external switch only activates the phrase function). Size: 11"L × 4"W × 8"H. Requires 3 AAA Batteries. Weight: 3/4 lb.

Operation:

1. Colt requires 3 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn Colt over and open up the Velcro enclosure, then using a Phillips head screwdriver remove the screw and lift the battery cover off. Observing proper (+) & (-) battery polarity, install 3 AA size batteries into the holder. Replace the battery compartment cover and screw.
3. **Set the ON/OFF switch to the ON position. Please Note: Due to switch adaption Colt works two ways. The first way is via your external switch.** Connect **Colt** to any external capability switch via the cord input jack. Make sure all connections are pushed in all the way, there should be no gaps. Your switch will do the same thing as pressing Colt's musical note on his ear. This is a short 5 to 7 second activation. If you activate your switch during activation it will pause. Activating your switch a second time will resume play. Great for teaching cause and effect and for developing tracking.

4. The second way is pressing the musical note on Colt's saddle (not switch adapted). Press the note and Colt trots to the lively theme song from The Lone Ranger ("William Tell Overture"). Run time is 30 seconds.
5. A second press of your switch will stop him. If you hit your switch again he will start back up from the beginning of the song.
6. Turn OFF Colt when finished using to prevent false activation and battery drain.

Troubleshooting:

Problem: Colt does not work.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Action #3: Make sure that you have plugged your switch in all the way. There should be no gaps.

Action #4: Try your external capability switches with another device to rule out the switches as the source of the problem.

Care of the Unit:

If you will not be using Colt for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Colt is surface washable only.